	Clinical Trial App Onboarding	MBL949A12201	Rev. B
---	--------------------------------------	--------------	---------------


Purpose:


The purpose of this document is to outline the steps taken to set-up the BIOSENSE app for the MBL949A12201 trial. Basic functionality of the app will be detailed in this document as well.

Setup:

Download the MBL949A12201 app from the apple/google play store. An email account must be created for each patient with the format -

MBL949A12201_yyyxxx@gmail.com - yyyy are the numbers for the respective sites (1001,1002,1003,1004) xxx would be for the specific patient number (e.g., 001, 002, 003).


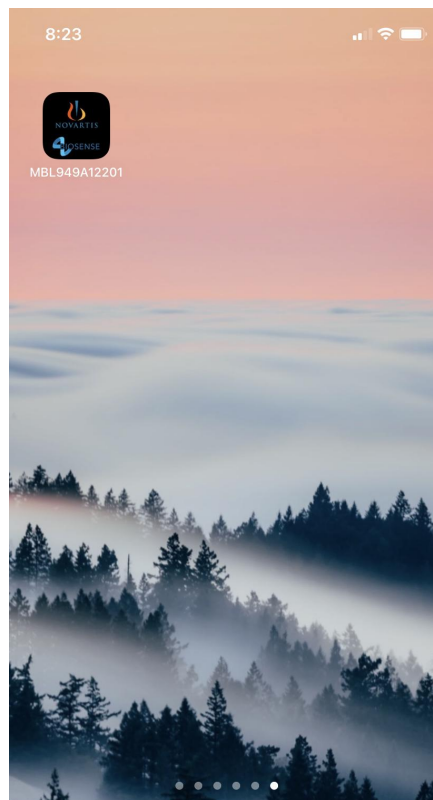



	Clinical Trial App Onboarding	MBL949A12201	Rev. B
---	--------------------------------------	--------------	---------------

Procedure For Test:

I. Onboarding

- A. After downloading the MBL949A12201 app from the apple/google play store, you will be prompted to select between two options.
 - 1. I have used this before.
 - 2. I have not used this before.
- B. Have the trial participant select “I have not used this before”.
- C. CRO/Site will supply the participant with the required email address for sign up. The trial participant with the site support will enter the MBL949A12201_1001xxx format under name and use the supplied email. The participant should enter a password they will remember in case they are somehow logged out of the account.

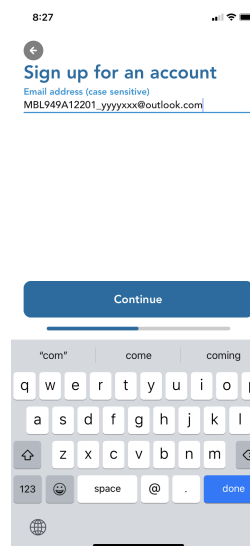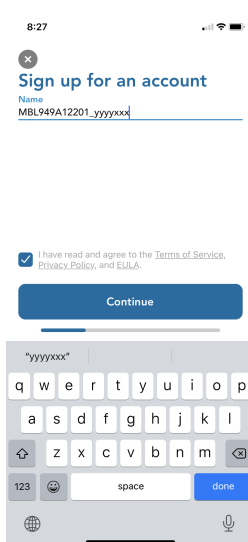


	Clinical Trial App Onboarding	MBL949A12201	Rev. B
---	--------------------------------------	--------------	---------------

D. After sign-up is complete, the CRO/Site must get the confirmation code from the email used and enter the confirmation code into the app.

E. The participant will automatically be taken to the main page of the app.


II. Main App Functionality

A. On the main screen, four grey circles are displayed with descriptions of when the measurement should be taken.

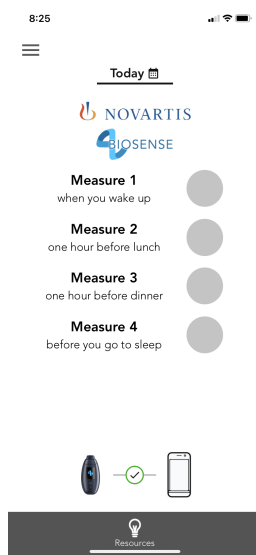


B. The device must be paired to the app to take a measurement. To do so they will click the **Red X** on the main page.




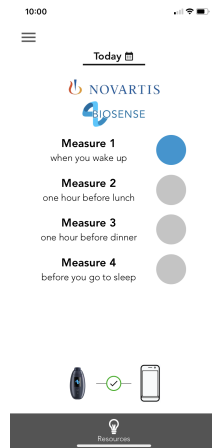
	Clinical Trial App Onboarding	MBL949A12201	Rev. B
---	--------------------------------------	--------------	---------------

- C. The BIOSENSE serial number will appear on the device list when the device is turned on. Turn on the device and click on the serial number on the app.

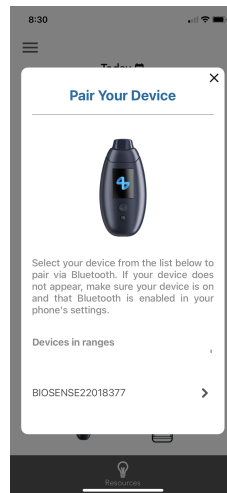

- D. Taking a measurement, based on MBL949A12201, will result in a circle being filled in with the color blue.
- E. To check how many measurements were taken on previous days, the user should click on the calendar at the top. The amount of blue circles under a day will indicate how many measurements were taken.
- F. The user should take 4 measurements over the course of a single day at these four times:
1. Measure 1: when you wake up
 2. Measure 2: one hour before lunch
 3. Measure 3: one hour before dinner
 4. Measure 4: before you go to sleep

	Clinical Trial App Onboarding	MBL949A12201	Rev. B
---	--------------------------------------	--------------	---------------



G. At the end of the day, all 4 circles will fill in blue. The following day, the circles will revert to grey and indicate that measurements still need to be taken for those time slots.

III. Troubleshooting

- A. If the trial participant runs into any issues they can access the “Resources” button below. This will take them to the BIOSENSE trial page to give an explanation of device functionality and answer any other questions they may have.
- B. Any questions that are not answered on the “Resources” page should be directed to the CRO/Site, who can either answer questions or communicate back to Readout.